

Create a test

1 Click on CREATE TEST



2 A dialogue box will appear:
Please scan cartridge barcode



3 Press the yellow button of the barcode scanner and scan the cartridge



Create a test

Once done, the below screen will appear



4

Fill in the Patient ID if available

5

Fill in the Sample ID

6

The module will be automatically selected
DO NOT CHANGE IT

Create Test

Patient ID: Имя пациента

Sample ID: № образца

Name: _____

Select Assay: Xpert MTB-RIF G3

Select Module: A1

Reagent Lot ID*: 02803 Expiration Date*: 2012/6/03

Test Type: Specimen

Sample Type: Other Other Sample Type: _____

Notes: _____

Start Test Scan Cartridge Barcode

7

Click on Start Test

8

A green light will blink from a given module: Open the door and place the cartridge



9

Fully close the module door, the test starts immediately
Wait 2 hour to see your result